


HelicotecUT[®] Plus

Assay the Urease Activity of helicobacter Pylori


- ▶ Skid proof, easy to grip.
- ▶ Hook-like tip, easy to transfer biopsy specimen.
- ▶ Blunt-point tip design, user's safety is considered.
- ▶ Customer-oriented design.
- ▶ All in one package.
- ▶ The best choice for real-time diagnosis and easy storage.

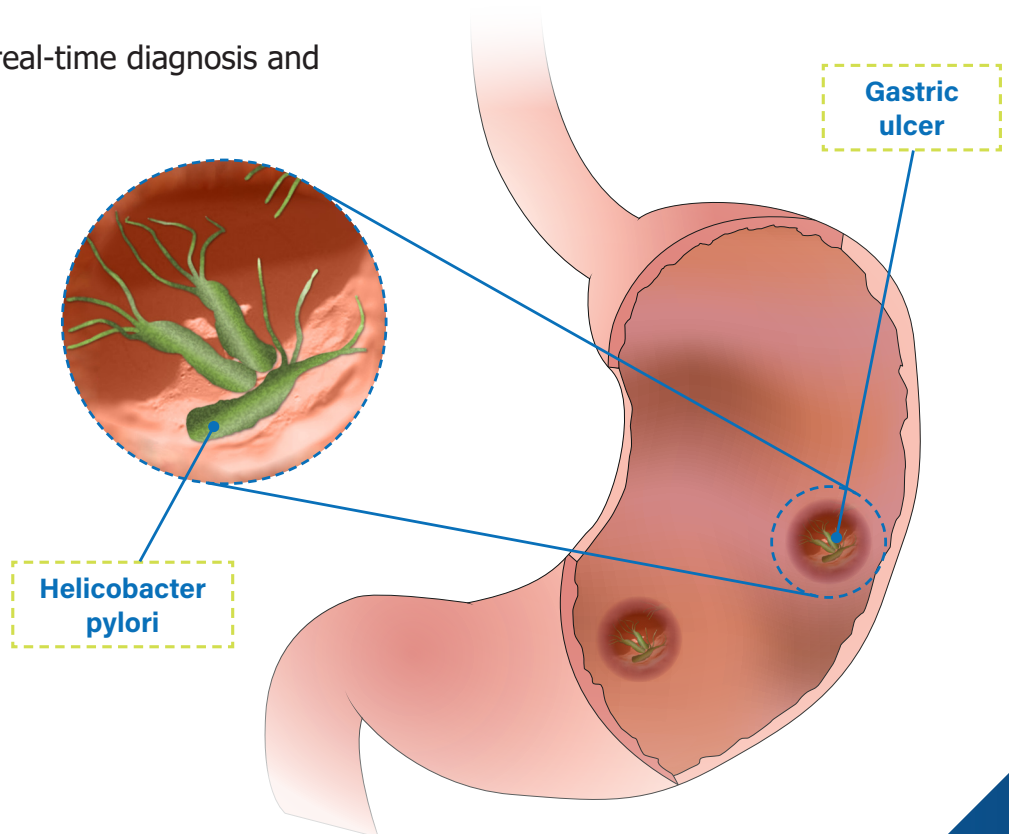

HelicotecUT® Plus

Reference	Description	QTY
HUP01	Paper Type Rapid Urease Test detects H. Pylori in gastric biopsy	50

Test in 4 steps:

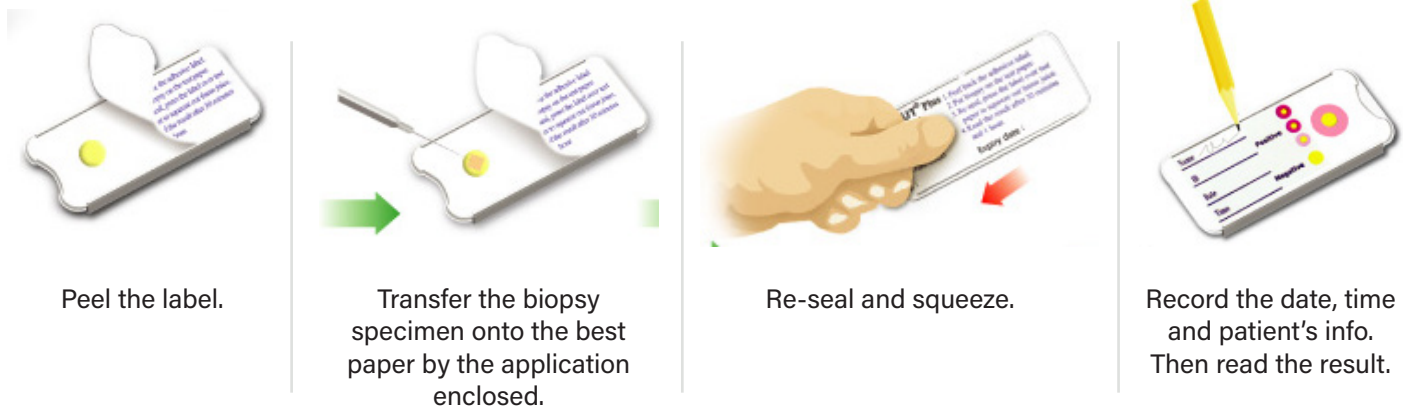

Clear result within 5 to 60 minutes

Reaction	5 mins	30 mins	60 mins	60 mins	60 mins	60 mins
Color of Ring	Magenta	Magenta	Magenta	Dark Pink	Light Pink	Yellow
Urease Activity	++++	++++	+++	++	+	-

Specification

HelicotecUT® Plus	1*	2*
Sensitivity	98,5%	100%
Specificity	92,5%	100%
Fastest positive result	5 mins	
Cut of time	60 mins	
Easier to record the patient's info and result on the same side by any kind of pen.		

The above data are according the clinical reports issued by 1* Kaohsiung Medical University hospital and 2* Taiped Medical University Hospital

Characteristics

- ▶ Clear, rapid and accurate result.
- ▶ High sensitivity and specificity.
- ▶ Applicators included for convenient usage.
- ▶ Superior quality.
- ▶ Favorable price.
- ▶ Easy to use, easy to store.



Version : HUT_EXP_1 Rev. 01 Date 04-07-23.

Creo Medical authorised local distributor

Creo Medical S.L.

Pol. Ind. Cordovilla, Calle D. N°1
31191 Cordovilla, Navarra (España)
Tel.: +34 948 293 324
E-mail: info.es@creomedical.com

